

## Nobel Biocare cold-worked titanium

In conjunction with its development of TiUnite<sup>®</sup>, Nobel Biocare collaborated with titanium suppliers to create a dental implant material with exceptional material properties, without needing to alloy.<sup>1</sup>

Since being introduced to the market, following extensive material and pre-clinical testing, Nobel Biocare's proprietary, cold-worked, commercially pure titanium and patented TiUnite surface have been documented as exceptionally strong<sup>2</sup> and clinically proven to enhance osseointegration<sup>3</sup> (Glauser et al 2001).

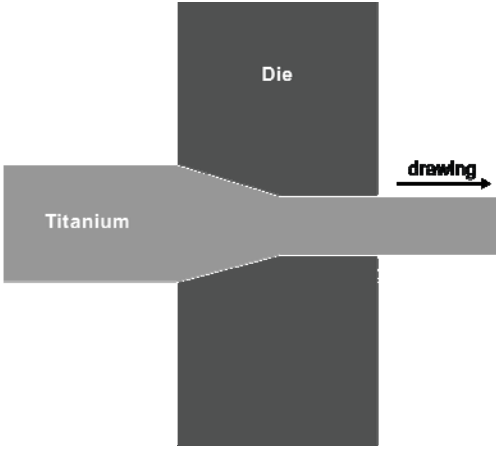
### Commercially pure standard titanium

Since his discovery of osseointegration, Prof PI Brånemark has prescribed commercially pure (c.p.) titanium<sup>4</sup> for use in dental implant restorations (Brånemark et al. 1985):

Key messages	Properties
<ul style="list-style-type: none"> <li>▪ One of the most documented and preferred biocompatible materials:               <ul style="list-style-type: none"> <li>– Material of choice for dental implants (Steinmann 1998)</li> </ul> </li> <li>▪ Nobel Biocare has used c.p. titanium for more than 20 years</li> </ul>	<ul style="list-style-type: none"> <li>▪ Highly biocompatible (Donachie 1988)</li> <li>▪ Excellent corrosion resistance (Donachie 1988)</li> <li>▪ Very low rate of adverse biological reactions (Bardos 1990)</li> <li>▪ Osseointegrates – integrates with living bone (Brånemark et al 1985)</li> </ul>

### Noble Biocare cold-worked titanium

Nobel Biocare commemorates the tenth year of using its high-performance titanium for its entire dental implant offering, which includes small diameters ( $\varnothing < 3.5$  mm):

Key messages	Cold-working process
<ul style="list-style-type: none"> <li>▪ Ten years of pre-clinical and clinical use</li> <li>▪ Proprietary cold-working (strengthening) process:               <ul style="list-style-type: none"> <li>– Significant gains in material strengths: yield, tensile and device fatigue</li> <li>– No need to sacrifice material purity through alloying<sup>1</sup></li> </ul> </li> <li>▪ Resolved material limitations of standard titanium for use with small diameter implants<sup>5</sup></li> <li>▪ Purity and strength confirmed in ongoing external material supplier and internal implant lot testing<sup>2</sup></li> </ul>	 <ul style="list-style-type: none"> <li>▪ At a specific temperature and drawing rate, titanium is deformed in a die, resulting in material strain hardening (Donachie 1988)</li> </ul>

<sup>1</sup> Alloying introduces elements that potentially cause adverse biological effects and can increase risk of fracture (elongation fracture)

<sup>2</sup> External material and internal implant lot testing confirms gains in material properties, data on file

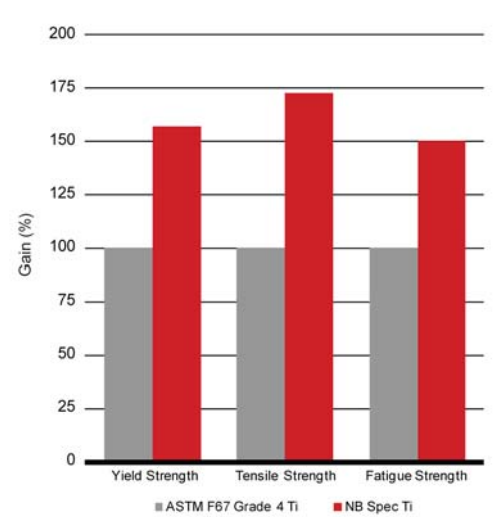
<sup>3</sup> Compared to implants with machined surfaces

<sup>4</sup> According to the American Society for Testing and Materials specification, ASTM F67

<sup>5</sup> Internal warranty system tracking NobelDirect 3.0 mm implants, data on file

## Substantial gains in titanium strength

Nobel Biocare's proprietary, cold-working process produces c.p. titanium with significant gains in material strengths:

<p><i>Key messages</i></p> <ul style="list-style-type: none"> <li>▪ Yield strength &gt;55%</li> <li>▪ Tensile strength (min.) &gt;70%</li> <li>▪ Device fatigue strength &gt;50%</li> </ul>	<p><i>Evidence</i></p>  <table border="1" style="margin-top: 10px; width: 100%; text-align: center;"> <thead> <tr> <th>Property</th> <th>ASTM F67 Grade 4 Ti (%)</th> <th>NB Spec Ti (%)</th> </tr> </thead> <tbody> <tr> <td>Yield Strength</td> <td>100</td> <td>~155</td> </tr> <tr> <td>Tensile Strength</td> <td>100</td> <td>~170</td> </tr> <tr> <td>Fatigue Strength</td> <td>100</td> <td>~145</td> </tr> </tbody> </table> <ul style="list-style-type: none"> <li>▪ Ongoing external material and internal implant lot testing confirms gains in material properties<sup>6</sup></li> </ul>	Property	ASTM F67 Grade 4 Ti (%)	NB Spec Ti (%)	Yield Strength	100	~155	Tensile Strength	100	~170	Fatigue Strength	100	~145
Property	ASTM F67 Grade 4 Ti (%)	NB Spec Ti (%)											
Yield Strength	100	~155											
Tensile Strength	100	~170											
Fatigue Strength	100	~145											

## Small diameter implant success

Nobel Biocare cold-worked titanium delivers excellent device fatigue strength and performance in small diameter implants ( $\varnothing < 3.5$  mm):

<p><i>Key messages</i></p> <ul style="list-style-type: none"> <li>▪ Nine years of small implant experience</li> <li>▪ No fatigue fracture for small diameter implants:<sup>7</sup> <ul style="list-style-type: none"> <li>– Effectively negates the need for a stronger material</li> </ul> </li> <li>▪ Small diameter implants:           <ul style="list-style-type: none"> <li>– Offer less invasive dental implant solutions for patients with narrow bone ridges, and limited space between teeth</li> <li>– Can minimize the need for guided bone regeneration (GBR) procedures</li> </ul> </li> <li>▪ Small diameter, one-piece implants are designed for restorations with exceptionally steep emergence angles for excellent final esthetics</li> </ul>	<p><i>Facts</i></p> <ul style="list-style-type: none"> <li>▪ Two-piece implant, launched in 2000:           <ul style="list-style-type: none"> <li>– Brånemark System Mk III <math>\varnothing 3.3</math> mm</li> </ul> </li> <li>▪ One-piece implant, launched in 2004:           <ul style="list-style-type: none"> <li>– NobelDirect <math>\varnothing 3.0</math> mm</li> </ul> </li> </ul>
--	--

<sup>6</sup> Data on file

<sup>7</sup> Internal warranty system tracking NobelDirect 3.0 mm implants, data on file

### *References*

Bardos DI. Titanium and Titanium Alloys: Medical and Dental Materials (Williams D ed.) Pergamon Press 1990:360-5.

Brånemark PI, Zarb GA, Albrektsson T, ed. Tissue-integrated prostheses: osseo-integration in clinical dentistry. Quintessence Publishers, Carol Stream IL, 1985:11-76, 129-143.

Donachie Jr, MJ (ed). TITANIUM: A Technical Guide. ASM International. ISBN 0871703092, 1988

Glauser R, Portmann M, Ruhstaller P, Lundgren AK, Hämmerle C, Gottlow J. Stability measurements of immediately loaded machined and oxidized implants in the posterior maxilla. A comparative clinical study using resonance frequency analysis. Appl Osseointegration Res 2001;2:27-9.

Steinemann SG. Titanium – the material of choice? Periodontology 2000 1998;17:7-21.