
# NobelClinician<sup>®</sup> Software

Installation guide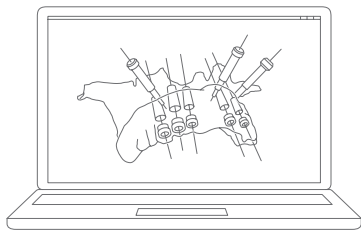

Please follow the instructions carefully.  
 Have the NobelClinician License Information document at hand.  
 Note: Your license is intended for one installation only.

## Installation of NobelClinician® Software

1. Check that your computer has access to the internet.
2. Check that your computer can view .pdf files (double-click on a .pdf file to test). Adobe® Reader® is an example of software typically used to read .pdf files.
3. Log on to your computer with a user account that has Administrator rights. We recommend that all other software on your computer is closed before the NobelClinician Software is installed
4. Insert the NobelClinician Installation DVD. If the install application does not start automatically, start the installer manually. If you have a 32-bit Windows® operating system, double-click on NobelClinician.exe. When working on a 64-bit Windows® operating system, double-click on NobelClinician(x64).exe.
5. A welcome message to the NobelClinician install wizard appears. Click "Next".
6. The NobelClinician End-User License Agreement is displayed. Read it carefully. You can also print this information. To accept the agreement, click on "I accept the terms in the license agreement", and click "Next". Otherwise, click "Cancel" and the install procedure will abort.
7. Choose a "destination folder" for the NobelClinician Software. This is the folder where the software files will reside. Then, click "Next".
8. Click on "Install" to start the actual installation.
9. When the software is installed, NobelClinician starts and a second installation wizard is shown, allowing to set up the software.
10. Enter the installation key that is printed on the NobelClinician Installation Information sheet. Click "Next".
11. When upgrading ProCMS to NobelClinician, a form is shown to enter some personal data, and the ProCera prefix of your ProCera Software Clinical Design. Enter the data, and click on "Next". The Upgrade Agreement is displayed. Read it carefully. You can also print this information. To accept the agreement, click on "I accept the terms in the license agreement", and click "Next". Otherwise, click "Cancel" and the install procedure will abort. A confirmation e-mail will be automatically sent to you.
12. Select the "data folder." This is the folder where the patient-related data will be stored. Please make sure that this folder resides on a hard disk with enough free disk space (A minimum of 5 GB of free disc space is needed. More disc space is used based on the number of patients. A typical dataset of one patient in NobelClinician is about 100 MB).

All users of the computer should have read/write access to this folder.

13. Select the preferred software language. This is the language used in the help files and the NobelClinician Assistant. Then, click "Next".
14. Now decide whether you want to install products from a local file or to download them from the server.
  - When you choose the local file, browse to the implant library file (with the extension .ILF) and click "Continue" to install all products in the file. The installation progress is shown.
  - If you choose to download the products from the server, a new dialog allows you to select the brands to be downloaded. Click "Continue" to start downloading. The download progress will be shown. It is possible to cancel the download process by returning to the previous page and switching to another option. When all products are installed, click "Finish".

The software is now started.
15. Log on as a specific user. For a first-time installation, enter the initial password, which is "sesam". Should you install an add-on license or a NobelClinician upgrade, enter your usual password. Then click on "Login".
16. The user registration form is shown. For a first-time installation, change the initial password by typing a new password (minimum 6 characters) and then typing the same new password again in the second field. In the Contact details, select the appropriate profile from the drop-down list. Optionally, adjust name, username and/or e-mail. Then click on "Next".
17. The NobelClinician End-User License Agreement is displayed. Read it carefully. You can also print this information. To accept the agreement, click on "I accept the terms in the license agreement", and click "Next". Otherwise, click "Cancel" and the installation procedure will abort.
18. Then choose the visibility of locally stored patient data.
  - Select the first option if you would like locally stored patient data to be visible to other users on the computer. This option is selected by default.
  - Select the second option if you would like the locally stored patient data to be hidden from other NobelClinician users on the computer.
19. Select the language to display the Instructions for Use and click on "Next".
20. Then click on "Finish". The Instructions for Use are now displayed in the chosen language and the software is ready for use.

A "NobelClinician" icon is now available on your desktop. Double-click on it to start the software. NobelClinician can also be started by clicking the Windows® Start button: browse the list of all programs that appears and click on Nobel Biocare. Then click on the NobelClinician icon to start the software. The Instructions for Use are also available here in a printable format.

At all times the License Agreement is available in the "About" dialog within the NobelClinician Software. It is advised to use the software while being connected to the internet for receiving software updates etc. An internet connection is required to prepare and send orders.

## Technical support

If you encounter any problems during the installation, please contact your local Customer Support (see page 54).

## For Mac OS® X.

Please follow the instructions carefully.

Have the NobelClinician License Information document at hand.

Note: Your license is intended for one installation only.

### Installation of NobelClinician® Software

1. Check that your computer has access to the Internet.
2. Insert the NobelClinician Installation DVD. If the install application does not start automatically, double-click the Apple® Disk Image file (.dmg) to mount it. The mounted NobelClinician disk will appear on your desktop.
3. Double-click this file. An Installation dialog appears, showing different options:
  - Install NobelClinician
  - Instructions for Use
  - End-User License Agreement
4. To install the NobelClinician Software, double-click the first option. A welcome screen is displayed. Click on “Continue”.
5. The NobelClinician End-User License Agreement is displayed. Read it carefully. You can also print this information.
6. Click “Continue”. A warning is displayed, asking for agreement with the End-User License Agreement.
7. Click “Agree” to continue with the installation. When clicking “Disagree” the installation procedure is aborted.
8. Next you can keep the default location for the installation (Applications) or you can change it to the location of your choice. This is the disk where the software files will reside.
9. Click “Install”.
10. Enter the username and password of an administrator account.
11. When the installation is finished, the software is opened and an installation wizard is shown.
12. Enter the installation key that is printed on the NobelClinician Installation Information sheet. Click “Next”.
13. When upgrading PCMS to NobelClinician, a form is shown to enter some personal data, and the Procera prefix of your Procera Software Clinical Design. Enter the data, and click on “Next”. The Upgrade Agreement is displayed. Read it carefully. You can also print this information. To accept the agreement, click on “I accept the terms in the license agreement”, and click “Next”. Otherwise, click “Cancel” and the install procedure will abort. A confirmation email will be automatically sent to you.
14. Now configure the software. Select the “data folder.” This is the folder where the

patient-related data will be stored. Please make sure that this folder resides on a hard disk with enough free disk space (approximately 100MB per patient file is needed). All users of the computer should have read/write access to this folder.

15. Next select the preferred software language. This is the language used in the NobelClinician help files and the Assistant. Then, click "Finish". The software starts.
16. Now decide whether to install products from a local file or to download them from the server (a stable, high-speed Internet connection is required).
  - When you choose the local file, browse to the implant library file (with the extension .ilf) on the DVD and click OK to install all products in the library. The installation progress is shown.
  - In case you choose to download the products from the server, a new wizard page allows to select the manufacturer(s) whose products are to be downloaded. Click OK to start downloading. The download progress is shown. It is possible to cancel the process by returning to the previous page and switch to another option.

When all products are installed, click Next.
17. Log on as a specific user.
18. You are prompted to change the password. Enter the received password (by e-mail) in the 'Change Password' dialog. Type a new password (minimum 6 characters). Then retype the new password. Click on "Save the password".
19. The NobelClinician End-User License Agreement is displayed. Read it carefully. You can also print this information. To accept the agreement, click on "I accept the terms in the license agreement", and click "Next". Otherwise, click "Cancel" and the install procedure will abort.
20. Then choose the visibility of locally stored patient data.
  - Select the first option if you would like locally stored patient data to be visible to other users on the computer. This option is selected by default.
  - Select the second option if you would like the locally stored patient data to be hidden from other NobelClinician users on the computer.
21. Select the language to display the Instructions for Use and click on "Next".
22. Then click on "Finish". The Instructions for Use are now displayed in the chosen language and the software is ready for use.

A NobelClinician icon is available in Applications (default location). Click on it to start the software.

At all times the License Agreement is available in the "About" dialog within the NobelClinician Software. It is advised to use the software while being connected to the Internet for receiving software updates etc. An Internet connection is required to prepare and send orders.

### **Technical support**

If you encounter any problems during the installation, please contact your local Customer Support (see page 54).

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