






# NobelReplace® Tapered Groovy, Replace Select™ Tapered and NobelGuide™

## INSTRUCTIONS FOR USE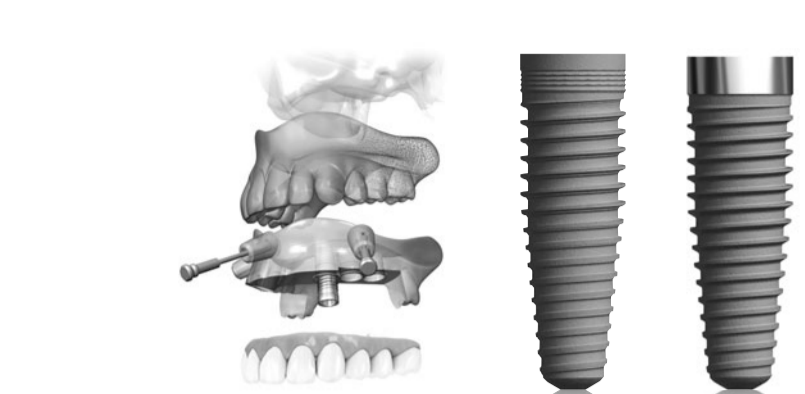

74029 A West 1110 Printed in Sweden. Date of issue 2011-07-12. All rights reserved. Nobel Biocare, the Nobel Biocare logo and all other trademarks used in this document are, if nothing else is stated or is evident from the context in a certain case, trademarks of Nobel Biocare. Product images in this folder are not necessarily to scale.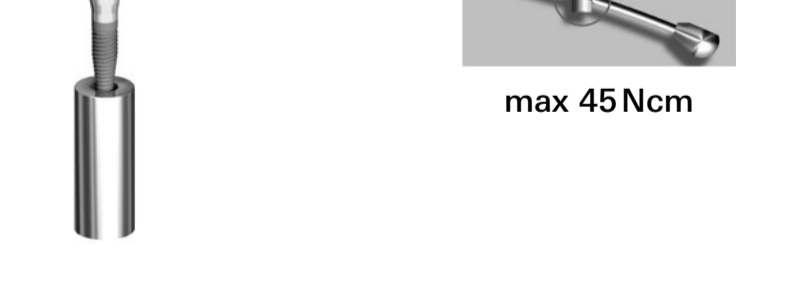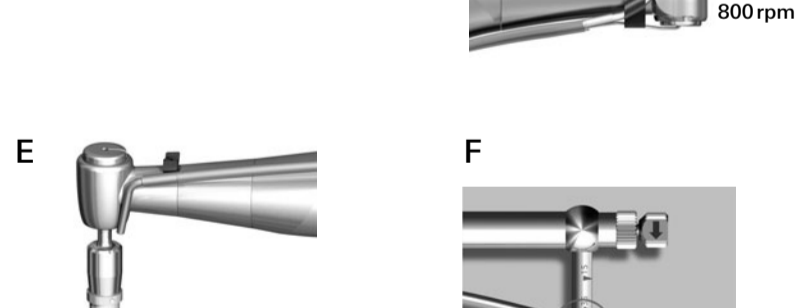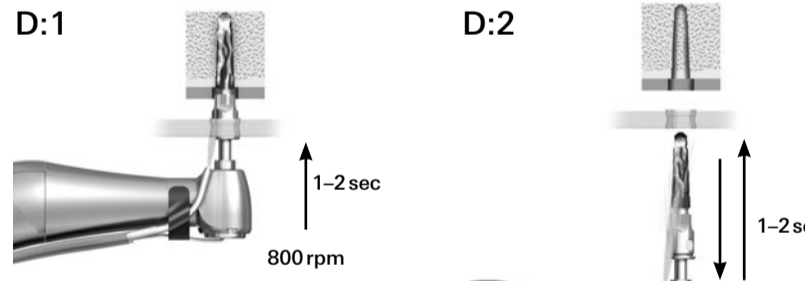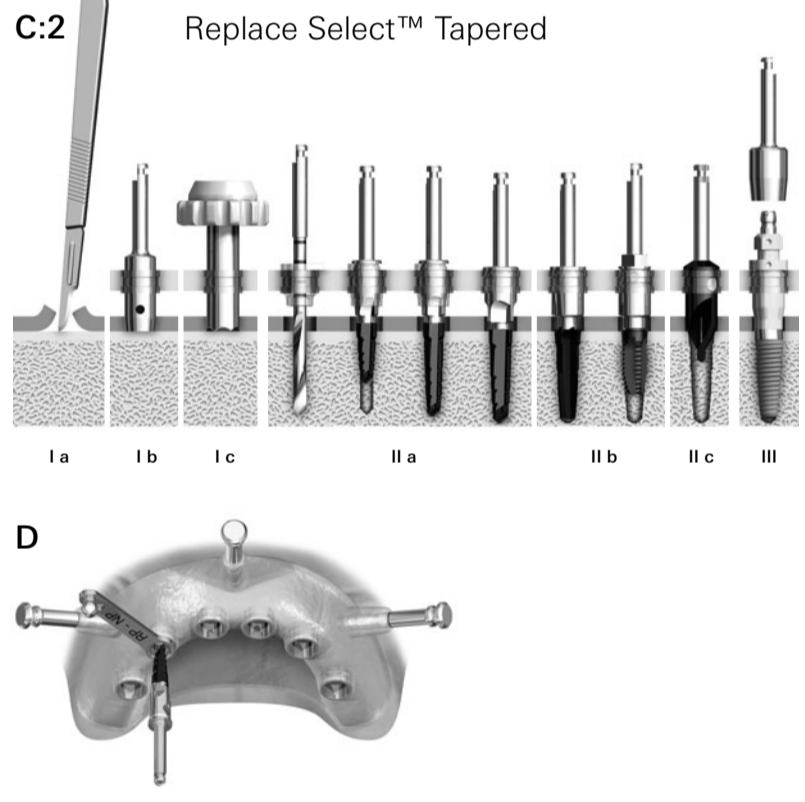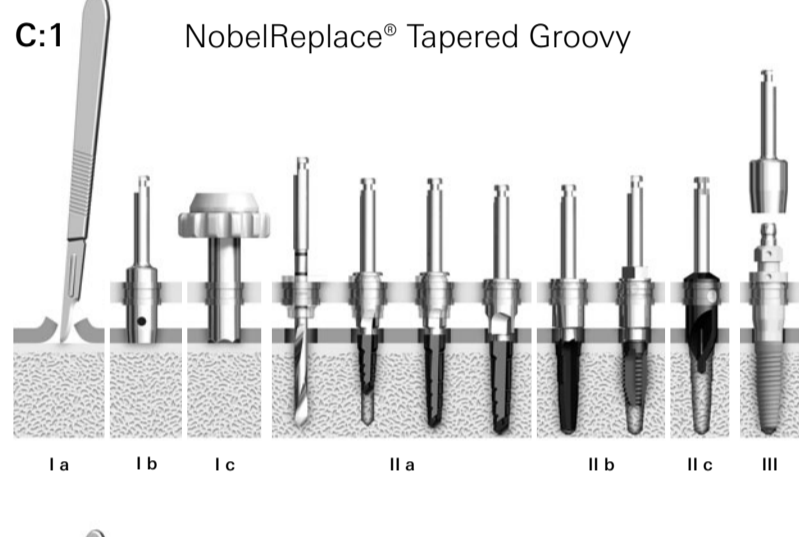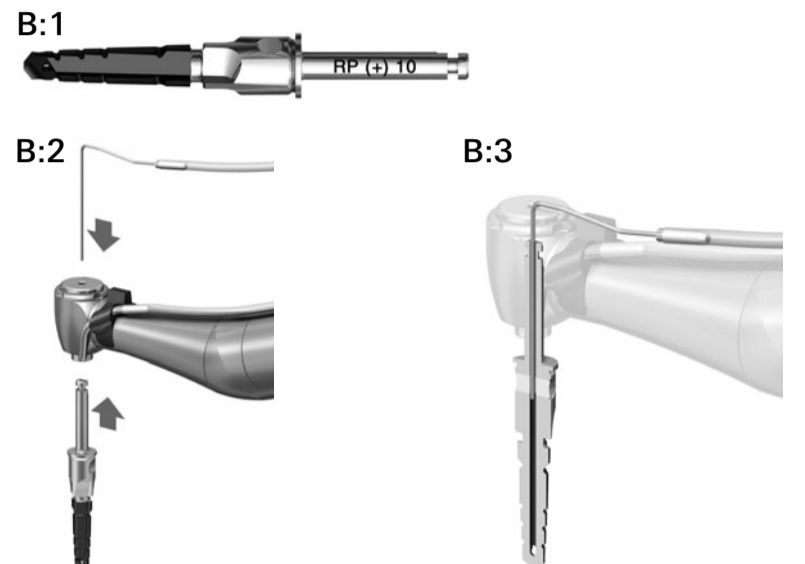

Platform	Platform diameter	Implant diameter	Lengths
NP	Ø 5.5mm	Ø 3.5mm	8mm, 10mm, 11.5mm, 13mm, 16mm
RP	Ø 4.3mm	Ø 4.3mm	8mm, 10mm, 11.5mm, 13mm, 16mm
WP	Ø 5.0mm	Ø 5.0mm	8mm, 10mm, 11.5mm, 13mm, 16mm
6.0	Ø 6.0mm	Ø 5.0mm	8mm, 10mm, 11.5mm, 13mm, 16mm*

\* 16mm implant is not available for Replace Select™ Tapered.

## ENGLISH

### Important: Please read.

**Disclaimer of liability:** This implant is part of an overall concept and may only be used in conjunction with the associated original components and instruments according to the instructions and recommendations of Nobel Biocare. Use of products made by third parties in conjunction with Nobel Biocare implants will void any warranty or other obligation, expressed or implied, by Nobel Biocare. The user of Nobel Biocare implants assumes the duty to determine whether or not any product is suitable for the particular patient and circumstances. Nobel Biocare disclaims any liability, expressed or implied, and shall have no responsibility for any direct, indirect, punitive or other damages, arising out of or in connection with any errors in professional judgment or practice in the use and surgical application of Nobel Biocare implants. The clinician is also obliged to study and comprehend the latest developments in regard to the Nobel Biocare implants and their applications. In cases of doubt, the clinician has to contact Nobel Biocare. Since the processing and surgical application of this product are under the control of the clinician, they are his/her responsibility. Nobel Biocare assumes no liability whatsoever for damage arising therefrom.

**Description:** Endosseous threaded dental implant made from biocompatible commercially pure grade 4 titanium with TiUnité™ surface.

**Indications:** Oral implants are to be integrated in the jaw bone (osseointegration) and used for anchoring or supporting tooth replacements in either jaw. Restorations range from a single tooth to a fixed/removable full dental arch or overdenture applications to restore chewing function.

**Contraindications:** It is contraindicated placing dental implants in patients: – who are medically unfit for an oral surgical procedure (see also below "Precautions/Warning – Pre-surgery"), – with inadequate bone volume unless an augmentation procedure can be considered. – in whom adequate sizes, numbers or desirable position of implants are not achieved to provide safe support of functional or eventually parafunctional loads.

**Instruction for clinician:** We strongly recommend that clinicians, new as well as experienced implant users, always go through special training before undertaking a new treatment method. Nobel Biocare offers a wide range of courses for various levels of knowledge and expertise. For more information please visit [www.nobelbiocare.com](http://www.nobelbiocare.com). Working the first time with a colleague, experienced with the new device/treatment method, will provide further insight and understanding. Nobel Biocare has a global network of mentors available for this purpose.

**General precautions/warnings:** One hundred percent implant success can never be guaranteed. Failure to observe the indicated limitations of use and working steps may result in failure. With respect to pediatric patients, routine treatment is not recommended until the end of the jaw bone growth has been properly documented. Pre-operative hard tissue or soft tissue deficits may yield a compromised esthetic and/or unfavorable implant angulation. It is strongly recommended that NobelReplace® Tapered Groovy and Replace Select™ Tapered implants are used only with Nobel Biocare surgical instruments and prosthetic components, as combining components that are not dimensioned for correct mating can lead to mechanical and/or instrumental failure, damage to tissue or unsatisfactory esthetic results.

**Precautions/Warning – Pre-surgery:** A careful clinical and radiological examination of the patient must be performed prior to surgery to determine the psychological as well as physical status of the patient. Special attention has to be given to patients who have local or systemic factors that could interfere with the healing process of either bone or soft tissue or the osseointegration process (e.g., cigarette smoking, uncontrolled diabetes, oro-facial radiotherapy, steroid therapy, infections in the neighboring bone). In general, implant placement and prosthetic design must accommodate individual patient conditions. In case of bruxism or unfavorable jaw relationships reappreciation of the treatment option may be considered.

**Precautions/Warning – At Surgery:** Particular caution should be taken when placing narrow platform implants in the posterior region due to risk of prosthetic overload. Lack of adequate quantity and/or quality of remaining bone, infection and generalized diseases may be potential causes for failure of osseointegration both immediately after surgery, or after osseointegration is initially achieved. Besides the mandatory precautions for any surgery such as aseptic handling, one must also be aware of potential damage to nerves and vessels by referring to anatomical knowledge and preoperative radiographs. Failure to recognize actual lengths of step/ twist drills relative to radiographic measurements or drilling beyond the depth intended can result in permanent injury to nerves or other vital structures, potentially resulting in permanent numbness to the lower lip and chin or leading to hemorrhage such as in the floor of the mouth. Implants may be tilted up to 45° relative to the occlusal plane. When used with angulations between 30° and 45°, the following applies: The tilted implants must be sited; A minimum of 4 implants must be used when using a fixed prosthesis in a fully dentulous arch. The clinician is advised to provide regular patient follow up and to inform about good oral hygiene.

**Tapered implant design:** NobelReplace® Tapered Groovy and Replace Select™ Tapered implants are tapered. The tapered implants are recommended for all bone qualities and have an internal tri-channel connection. NobelReplace® Tapered Groovy implants have TiUnité™ surface on top of the platform. Replace Select™ Tapered implants have TiUnité™ surface with a 1.5mm high machined collar.

**Tapered drills:** Tapered Drills are internally irrigated allowing for additional cooling. A specific drilling technique should be applied to prevent irrigation holes becoming plugged with bone. During drilling use an in-and-out motion and drill in bone for 1–2 seconds. Move the drill up without stopping handpiece motor which allows the irrigation to flush away debris. Reusable Tapered Drills and Screw taps should be replaced approximately every 200 uses or earlier when cutting efficiency declines. Tapered drills are unique for each implant length. For cleaning and sterilization of tapered drills see product specific instructions for use for tapered drills or consult "Dental Professionals" section available at [www.nobelbiocare.com](http://www.nobelbiocare.com).

### Surgical procedures

**Mounting of the surgical template:** The radiographic guide was held in place by a radiographic (beta) index during the CT scan. The surgical template must be placed with maximum attention to find the exact same position that the radiographic guide had during the CT scan. The use of a lab-produced surgical index is mandatory. An adequate number of anchor pins (using the Guided Twist Drill Ø1.5mm) must be placed with strategic positioning and orientation to secure the surgical template in the correct position. Maximum attention must be paid to secure that the surgical template does not move in any direction from the correct position when being manipulated with instruments (e.g. lateral shift through inadequate handling of pilot/drills in "knife-edge ridge" situations or shift/dislodgement of surgical template due to excessive vertical force application during implant installation). In situations where two or more neighbouring implants are placed, regardless if it is a free-end situation or a situation with one or more distal teeth for support of the surgical template, it is recommended to use at least one anchor pin in this area. If necessary, place implants in a staggered approach.

**Surgical access options (C1/C2 I a or C1/C2 II b or C1/C2 I d):** Flapless procedure: If a flapless procedure is chosen, it is recommended to use a soft tissue punch before any other instruments are used to generate a clean cut (C1/C2 I b or C1/C2 I d). The surgical template is temporarily detached after punching to carefully remove the punched tissue. The surgical template is repositioned carefully in the same surgical position and orientation to compensate for the height of the surgical template and drill guide (B 1). The counterbore/start drill (single use) is recommended to be used as a countersink (maximum 800rpm) at the end of the drilling procedure (C1/C2 II c), to create adequate access for the guided implant mount. Thorough irrigation/finishing of the site is advised before the implant is placed.

**Non-flapless procedure (flap, mini-flap) (C1/C2 I a):** Following the procedure described in the concept manual, keratinized tissue may be saved and also submerged placing of the implant is possible allowing for simultaneous bone augmentation procedures etc.) by using a mini-flap or flap. Nevertheless the counterbore/start drill (C1/C2 II c) should be used at maximum 800rpm with profuse irrigation before the implant is placed to ensure adequate access. Be aware that the counterbore/start drill is not intended for use as a countersink.

- During drilling procedures bone quality should be considered (please see A recommended drill sequences based on bone quality to ensure optimal primary stability when applying Immediate Function). **Caution:** The Guided Tapered Drills are identified by the "+" designation on the shaft. The inbuilt depth stops on the Guided Tapered Drills correspond to the 8, 10, 11.5, 13 and 16mm implants. This indicates the tapered drills are 8mm longer to compensate for the height of the surgical template and drill guide (B 1). Drill extends up to 1mm longer than the implant when seated. Allow for this additional length when drilling near vital structures (the yellow safety zone in the NobelGuide™ protocol includes the extended drill lengths). Figure C1/C2 shows irrigation access for the RP 13mm implant (I a medium bone / I b dense bone protocol) see also section (B) below. Apply the Guided Start Drill/Counterbore for the Replace System (II c) if sub-crestal placement was performed to provide adequate access for the Guided Implant Mount.
- Prepare implant site (D) according to sequence described in (A). Drilling must proceed at high speed (maximum 800rpm) under constant and profuse irrigation by sterile saline at room temperature. An in-and-out motion, over the complete extent of the osteotomy, is needed when preparing the site to avoid overheating (D1, D2).
- Open the implant packaging and mount the guided implant mount onto the implant and pick up the entry from inner casing. The implants are ideally installed with low speed, max 25rpm, using a drilling machine (E) or by use of Manual Torque Wrench Surgery.
- Place and tighten the implant using maximum **45Ncm** installation torque (F). **Caution:** Never exceed insertion torque of **45Ncm**. Overtightening an implant may lead to damage of the implant, fracture or necrosis of the bone site.
- If the implant gets stuck during implant installation or **45Ncm** is achieved before fully seated, rotate the implant counter clockwise using drilling machine or manual torque wrench, in reverse mode, and remove implant from site. Replace the implant back into titanium casing before proceeding further. For Immediate Function, the implant should be able to withstand a final torque of **45Ncm**.
- Dense bone protocol: Dense Bone Drill** in conjunction with the **Guided Screw Tap** should be used in hard bone situations when the implant can not be fully seated (C1/C2 II b). a. For 13mm and 16mm long implants, select the dense bone drill matching the diameter and length of the implant. b. Select the screw tap matching the diameter of the implant. c. Place the screw tap into prepared implant site using low speed 25rpm. d. Screw tap to appropriate length. e. Switch the handpiece to reverse mode and back the screw tap up. Continue with implant installation until desired position is achieved using max **45Ncm** installation torque. 7. Depending on surgical protocol of choice, place a cover screw or abutment (G) and sutures, if indicated. 8. See figure (1) for NobelReplace® Tapered Groovy and Replace Select™ Tapered implant data.

For additional information on the NobelGuide™ concept and the surgical procedures please consult the NobelGuide™ Concept Manual available at [www.nobelbiocare.com](http://www.nobelbiocare.com) or request latest printed version from a Nobel Biocare representative.

**Sterility:** All implants and step/twist drills are delivered sterile, and are for single use only prior to the labeled expiration date. Do not use implants if the packaging has been damaged or previously opened. For steam sterilization of surgical kit, sterilize at 135°C/274°F, for 5 minutes.

**Adverse events:** Treatment by means of implants may lead to loss of bone, biological or mechanical failures including fatigue fracture of implants.

**Manufacturer:** Nobel Biocare AB, Box 5190, SE-402 26 Göteborg, Sweden. Phone: +46 31 81 88 00. Fax: +46 31 16 31 52. [www.nobelbiocare.com](http://www.nobelbiocare.com)

**Caution:** The caution text "Federal (USA) law restricts the sale of this device to, or on the order of, a licensed physician or dentist" is shown on labels with "Rx Only". Please note that not all products may have been licensed in accordance with Canadian law.

### A NobelReplace® Tapered System

The NobelReplace® Tapered System is designed for ease of use for all bone densities. The drilling protocol for NobelGuide™ follows the original free-hand drilling sequence. In addition to this protocol, just after the 2mm twist drill, the tapered 8mm NP drill has to be used. This drill is guided before engaging the bone and provides guidance for the longer NP drill (if an implant longer or wider than NP 8mm is placed). For reasons of drilling precision, the step using the 8mm drill is mandatory and must not be skipped.

In dense bone situations, both the dense bone drill and the guided screw tap are to be used if insertion torque exceeds **45Ncm**.

The Guided Twist Drill 2mm, all tapered drills including the start drill/counterbore are designed for internal irrigation (internal opening through the top of the drill towards the tip to be connected with compatible counter-angles).

The drilling protocol is also printed on the Guided Surgery Kit Box.

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