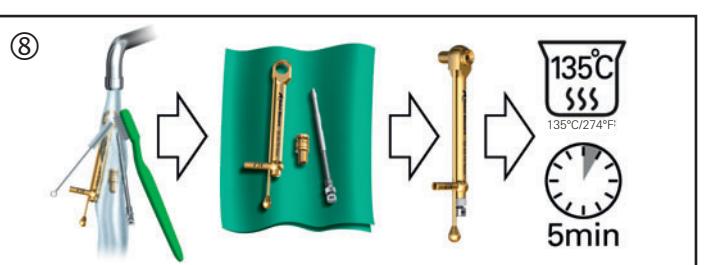
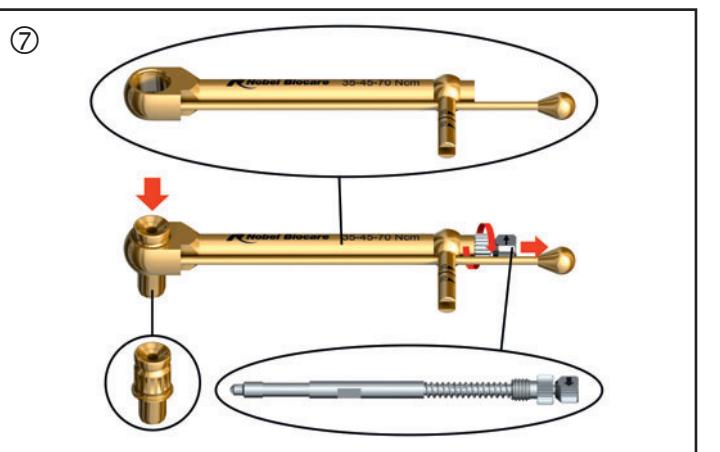
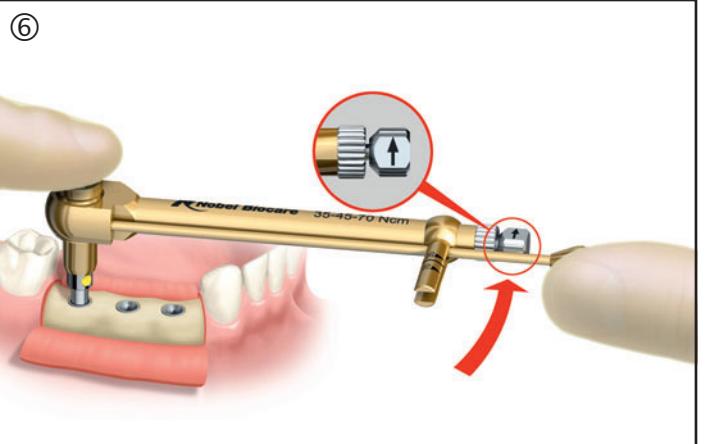
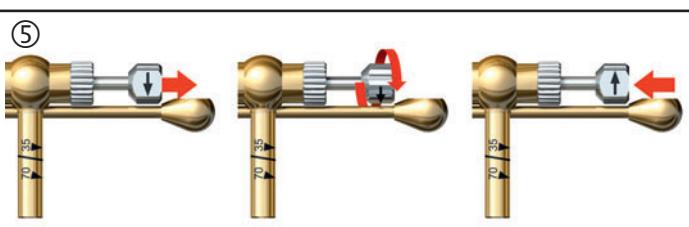
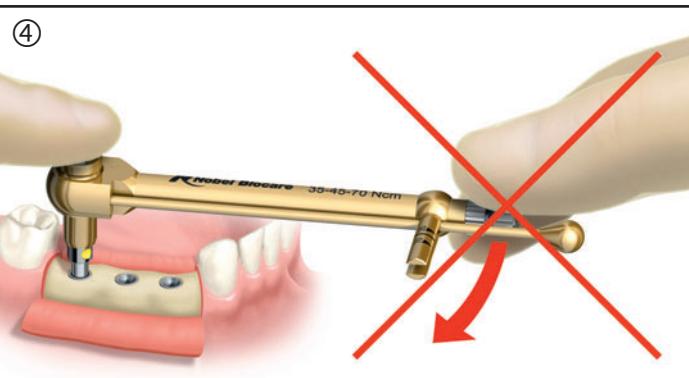
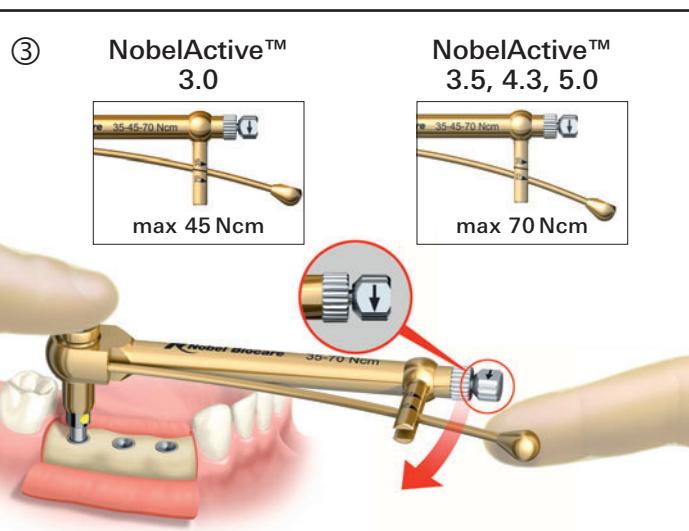
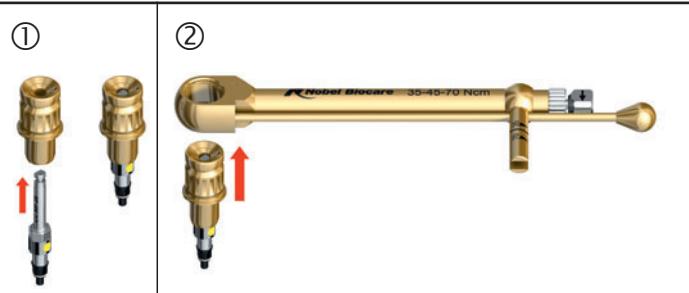
NobelActive™ Manual Torque Wrench

Instructions for use – Surgical Application

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GB

Instructions for use – Surgical application

Indications: The surgical Manual Torque Wrench is intended to be used for placement or removal of implants the day of implant insertion and for final implant tightening.

Warning: The use of the wrench body instead of the level arm may result in excessive torque (Fig. 4). This may excessively compress the bone and result in bone resorption.

Handling procedures: Insert the Implant Driver Conical Connection (Fig. 1, 2).

For correct handling and tightening torque, see the Instructions for use for NobelActive™ implants (Fig. 3). The maximum tightening torque

for NobelActive™ 3.0 implants is 45 Ncm and for NobelActive™ 3.5, 4.3 and 5.0 70 Ncm.

Warning: The use of the wrench body instead of the level arm may result in excessive torque (Fig. 4). This may excessively compress the bone and result in bone resorption.

The tool may also be used for adjustment of implant position during placement.

If necessary, the implant can be backed out using the wrench with the direction indicator arrow pointing counterclockwise (Fig. 5, 6).

Disassemble all parts and clean them thoroughly. Allow them to dry completely before sterilizing (Fig. 7, 8).

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