Handpiece Zygoma 20:1

DE	Gebrauchsanweisung
DK	Brugervejledning
ES	Instrucciones de uso
FI	Käyttöohjeet
FR	Mode d'emploi
GB	Instructions for use
GR	Οδηγίες χρήσης
IT	Istruzioni per l'uso
NL	Gebruiksaanwijzing
PT	Instruções para utilização
SE	Bruksanvisning
CZ	Pokyny k použití
EE	Kasutusjuhised
HU	Használati útmutató
LT	Naudojimo instrukcijos
LV	Instrukcijas lietotājam
MT	Struzzjonijiet għall-użu
PL	Instrukcja użytkowania
SK	Návod na použitie
SI	Navodila za uporabo

Contents

DE	Gebrauchsanweisung	4
DK	Brugervejledning	6
ES	Instrucciones de uso	8
FI	Käyttöohjeet	10
FR	Mode d'emploi	12
GB	Instructions for use	14
GR	Οδηγίες χρήσης	16
IT	Istruzioni per l'uso	18
NL	Gebruiksaanwijzing	20
PT	Instruções para utilização	22
SE	Bruksanvisning	24
CZ	Pokyny k použití	26
EE	Kasutusjuhised	28
HU	Használati útmutató	30
LT	Naudojimo instrukcijos	32
LV	Instrukcijas lietotājam	34
MT	Struzzjonijiet għall-użu	36
PL	Instrukcja użytkowania	38
SI	Návod na použitie	40
SK	Navodila za uporabo	42
	All figures	45

Technische Daten, Abb. 1

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Technical Data, Fig. 1

Handpiece Zygoma 32615
 Drive Speed: Max, 40,000 rpm
 Transmission: 20:1
 Maximum torque rotary instrument: 50 Ncm

Recommended Instrumentation and Tooling Electronic Equipment and Machinery

OsseoSet 100

Use program 2 or 3 for drilling at maximum 2000 rpm.
 Use program 4 for implant installation limited to 50 Ncm.

Recommendations for using surgical units other than OsseoSet 100

The possibility to adjust the gear mechanism of the contra-angle handpiece to 20:1 ratio.
 Maximum drill speed limited to 2000 rpm.

Maximum speed for implant installation limited to 45 rpm.
 Maximum torque for implant installation limited to 50 Ncm.

Note: Do not use with parts from any other sources. Any manipulation is impermissible and may result in damage or injury. Handpiece Zygoma is a medical product according to the applicable national provisions of law.

Recommended Tooling

Handpiece tools

Use contra-angle burs with a shank diameter of 2.35 mm as per DIN EN ISO 1797-1 standard made of steel or hardened metal, incl. Zygoma drills with
 – a total length of max. 120 mm
 – a cutting diameter of up to 4.5 mm can be used.

Connection of the tool, Fig. 2

Open up the locking ring completely (see Fig. 2A). Insert the tool and rotate it slightly until it falls into the locked position in the chuck. Turn the locking ring back completely (see Fig. 2B). Check that the tool is properly in place by pulling it.

Removing the tool

Open up the locking ring completely (see Fig. 2A). Remove the tool from the shank. Turn the locking ring back completely (see Fig. 2B).

Note: Never fit or remove tools when the motor is running. Be careful when touching sharp tools. Use gloves when testing, inserting and removing tools in order to avoid any risk of injury and infection. Never set in motion with the locking ring in open (see Fig. 2A). Tools which deviate from the stated measurements must not be used. Never use tools with worn-down shafts as they may work loose during treatment, possibly causing injury.

Cleaning & Sterilization

The Handpiece Zygoma should be cleaned and lubricated immediately after use. The handpiece should be handled in the same way as surgical handpieces.

Exterior Cleaning

The Handpiece Zygoma can be cleaned under running water, or with an alcohol solution. Do not place the handpiece in an ultrasonic cleaner as this may result in malfunction. If other cleaning agents are to be used, first consult the respective manufacturers' instructions.

Lubrication, Fig. 3

Use a suitable lubricant spray that meets the local regulatory requirements. Lubricate Handpiece Zygoma after each surgery and before autoclaving. Insert the handpiece in the nozzle and spray for about one second. Hold the handpiece firmly against the nozzle during spraying.

Disinfection

Dry the outer surfaces with gauze soaked in alcohol solution. Then dry the handpiece completely.

Note: *Do not place the handpiece in disinfectant baths as this may result in malfunction.*

Packaging

Pack the Handpiece Zygoma in a sterilization bag. In order to prevent oil seeping through, the handpiece can be wrapped in 3x3 cm gauze and packed in double wrapping.

Sterilization, Fig. 4

Steam autoclaving (vacuum or gravity) at max 135° C, minimum 15 minutes, is recommended, with the same procedure used for other surgical instruments (see the recommendations for the autoclaving unit used). Technically, the instrument can also withstand chemiclavng. Follow the drying routines carefully for the autoclaved material and remove the handpiece from the autoclave immediately after each cycle to prevent the risk of contact corrosion.

Note: *Do not continue to use in the event of damage, irregular operating noise, excessive vibration, excessive tool runout or inadequate tool clamping.*

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Hospimedics S.A.