
Healing Abutments

INSTRUCTIONS FOR USE

74389 C West 1304 Printed in Sweden. Date of issue 2011-12-09.
All rights reserved. Nobel Biocare, the Nobel Biocare logotype and all other trademarks used in this document are, if nothing else is stated or is evident from the context in a certain case, trademarks of Nobel Biocare. Product images in this folder are not necessarily to scale.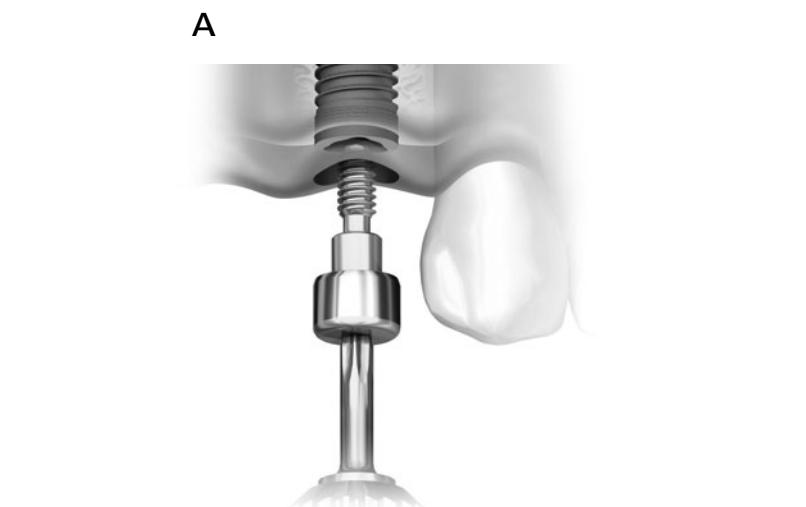

ENGLISH

Important: Please read.

Healing Abutments

Internal conical connection for NobelActive® and NobelReplace® CC.
Internal tri-channel connection for the following implant systems: NobelReplace®, Replace Select™ and NobelSpeedy® Replace.

External hex connection for Bränemark System® and NobelSpeedy® Groovy.

Disclaimer of liability: This product is part of an overall concept and may only be used in conjunction with the associated original products according to the instructions and recommendation of Nobel Biocare. Non-recommended use of products made by third parties in conjunction with Nobel Biocare products will void any warranty or other obligation, express or implied, of Nobel Biocare. The user of Nobel Biocare products has the duty to determine whether or not any product is suitable for the particular patient and circumstances. Nobel Biocare disclaims any liability, express or implied, and shall have no responsibility for any direct, indirect, punitive or other damages, arising out of or in connection with any errors in professional judgment or practice in the use of Nobel Biocare products. The clinician is also obliged to study the latest developments in regard to the Nobel Biocare products and their applications regularly. In cases of doubt, the clinician has to contact Nobel Biocare. Since the processing and surgical application of this product is under the control of the clinician, they are his/her responsibility. We assume no liability whatsoever for damage arising thereof.

Description: A premanufactured endosseous dental implant abutment directly connected to the endosseous dental implant intended for use as a temporary aid in prosthetic rehabilitation.

Indications: To be used as a transmucosal extension secured to the implant during healing of intra-oral soft tissue following insertion of the implant into the bone or after second-stage surgery. For internal conical connection implants a specific Healing Abutment Bridge is available. The healing abutment is specially designed to avoid any bone to grow on platform and by that prepare for the specially designed bridge impression coping. Using the series of components facilitates the treatment and prepare for an implant level bridge.

Instruction for clinicians: We strongly recommend that clinicians, new as well as experienced implant users, always go through special training before undertaking a new treatment method. Nobel Biocare offers a wide range of courses for various levels of knowledge. For more info visit www.nobelbiocare.com. Always work with an experienced colleague the first time you employ a new treatment method. Nobel Biocare has a global network of mentors for this purpose.

Precautions: It is imperative that the implant platform is free from tissue remnants before seating the abutment to avoid any remaining gap.

Procedural Precautions: Complete seating of the component on the endosseous implant is essential to avoid loosening. Because of the small size of prosthetic components, care must be taken that they are not swallowed or aspirated by the patient.

Sterility: Healing abutments are delivered sterile and if not used, could be re-sterilized for 5 minutes at 135°C/274°F using steam sterilization.

Note: Use of non-sterile abutments may lead to infection of tissues or infectious diseases.

Magnetic resonance environment: Nobel Biocare abutments have not been evaluated for safety and compatibility nor have they been tested for heating or migration in the magnetic resonance environment.

Handling procedures

Clinical Procedure

1. Select appropriate healing abutment.
2. Connect to implant and tighten using Unigrip™ Screwdriver (A).

For additional information on restorative and dental laboratory procedures please consult treatment guidelines available at www.nobelbiocare.com or request latest printed version from a Nobel Biocare representative.

Materials: Healing Abutments: Titanium alloy 90% Ti, 6% Al, 4%V.

Manufacturer: Nobel Biocare AB, Box 5190, 402 26
Västra Hamngatan 1, 411 17 Göteborg, Sweden
Phone: +46 31 81 88 00. Fax: +46 31 16 31 52. www.nobelbiocare.com

Caution: The caution text "Federal (USA) law restricts the sale of this device to, or on the order of, a licensed physician or dentist" is shown on labels with "Rx Only".

Please note that not all products may have been licenced in accordance with Canadian law.

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