


# Surgical protocol

Nobel Biocare N1™ system

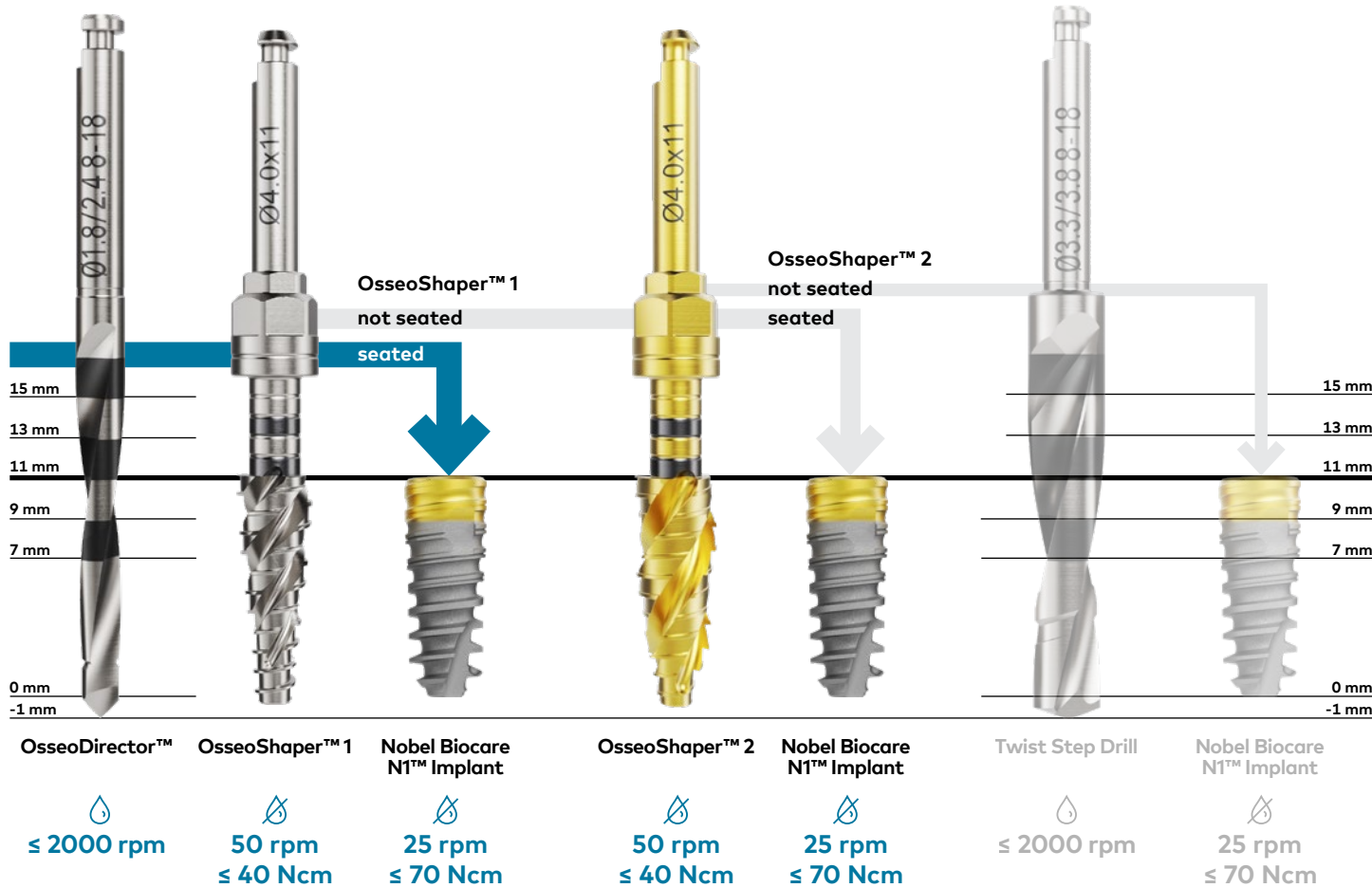

## Implant orientation



## Implant driver orientation



# Tightening torques

Nobel Biocare N1™ system



Nobel Biocare N1 Base

**20 Ncm**



Screwdriver Nobel Biocare N1 Base

Healing and impression taking components

**Hand-tighten**



Screwdriver Omnigrip Mini

Temporary and final abutments

**20 Ncm**



Screwdriver Omnigrip Mini

Temporary and final abutments

**20 Ncm**




Screwdriver Multi-unit for straight Multi-unit Abutment

**Implant level**



**Base level**

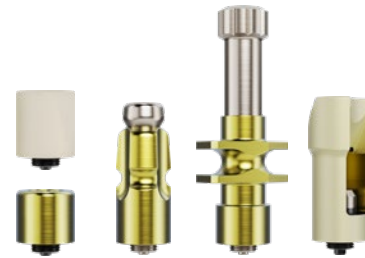

**Multi-unit Abutment**



\*At Multi-unit Abutment level the tightening torque is 15 Ncm. Depending on the type of restoration, a Unigrip or Omnigrip mini screwdriver is used in order to connect the restoration with the abutment.