

creos™ mucogain

Scientific evidence
September 2018

creos™ mucogain:

Overview

creos mucogain is a resorbable collagen matrix composed of highly purified porcine collagen and elastin fibers

- Open interconnecting porous structure^{1,2,3}
- Mechanical strength^{3,4}
- Memory effect³
- Soft tissue regeneration^{4,5}
- No biological complications in initial clinical use^{4,5}

1. Heschel I, et al. 2002. US patent 6,447,701 B1.

2. Boekema B, et al. J Mater Sci Mater Med 2014;25(2):423–433.

3. Olde Damink L, et al. 52nd Annual Conference of the German Society for Biomedical Engineering; Sep 26-28, 2018; Aachen.

4. Wessing B, Vasilic N. Clin Oral Implants Res 2014;25(s10):342.

5. Montero E, et al. 27th Annual Scientific Meeting of the European Association for Osseointegration; Oct 11-13, 2018; Vienna.

Note: mucomaix® (Matricel GmbH) is distributed as creos mucogain since October 2018.



creos™ mucogain: open interconnecting porous structure


- Open interconnecting porous structure produced by a patented process¹
- Designed to promote soft-tissue regeneration through the migration of cells and blood vessels into the matrix^{2,3}

1. Heschel I, et al. 2002. US patent 6,447,701 B1.

2. Boekema B, et al. J Mater Sci Mater Med 2014;25(2):423–433.

3. Olde Damink L, et al. 52nd Annual Conference of the German Society for Biomedical Engineering; Sep 26-28, 2018; Aachen.

Note: mucomaix® (Matricel GmbH) is distributed as creos mucogain since October 2018.

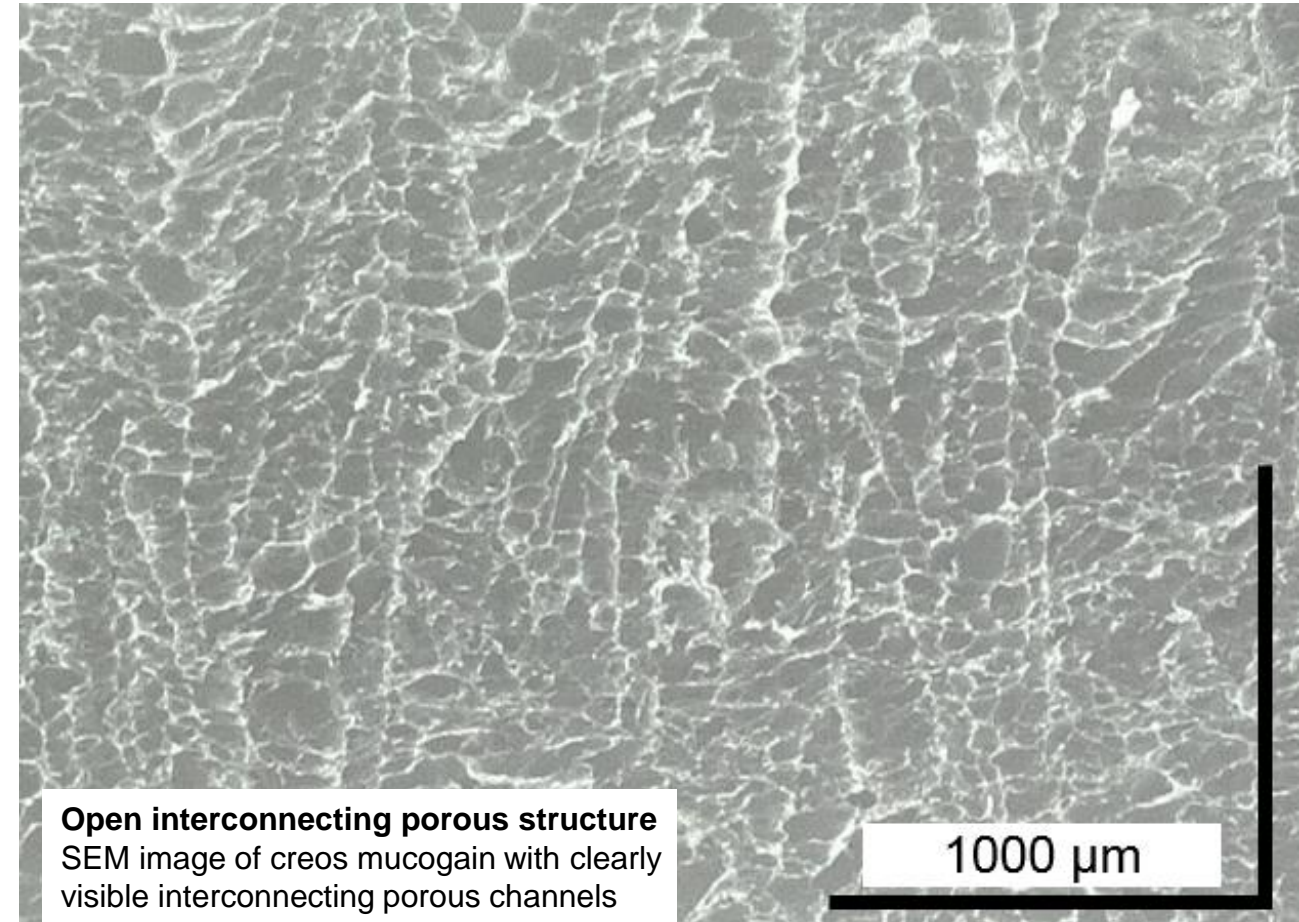

Open interconnecting porous structure
SEM image of creos mucogain with clearly visible interconnecting porous channels

Courtesy of Matricel GmbH, Herzogenrath, Germany

creos™ mucogain: mechanical strength

- High suture retention when dry and hydrated to enable fixation with sutures²
- Suture pull-out strength is sufficient for tunneling technique¹

1. Wessing B, Vasilic N. Clin Oral Implants Res 2014;25(s10):342.

2. Olde Damink L, et al. 52nd Annual Conference of the German Society for Biomedical Engineering; Sep 26-28, 2018; Aachen.

Note: mucomaix® (Matricel GmbH) is distributed as creos mucogain since October 2018.




Stress resistance for easy handling

Courtesy of Bastian Wessing

creos™ mucogain: memory effect

- When hydrated, the initial volume of the graft is maintained after repeated compression¹

Volume maintenance upon repeated compression in vitro

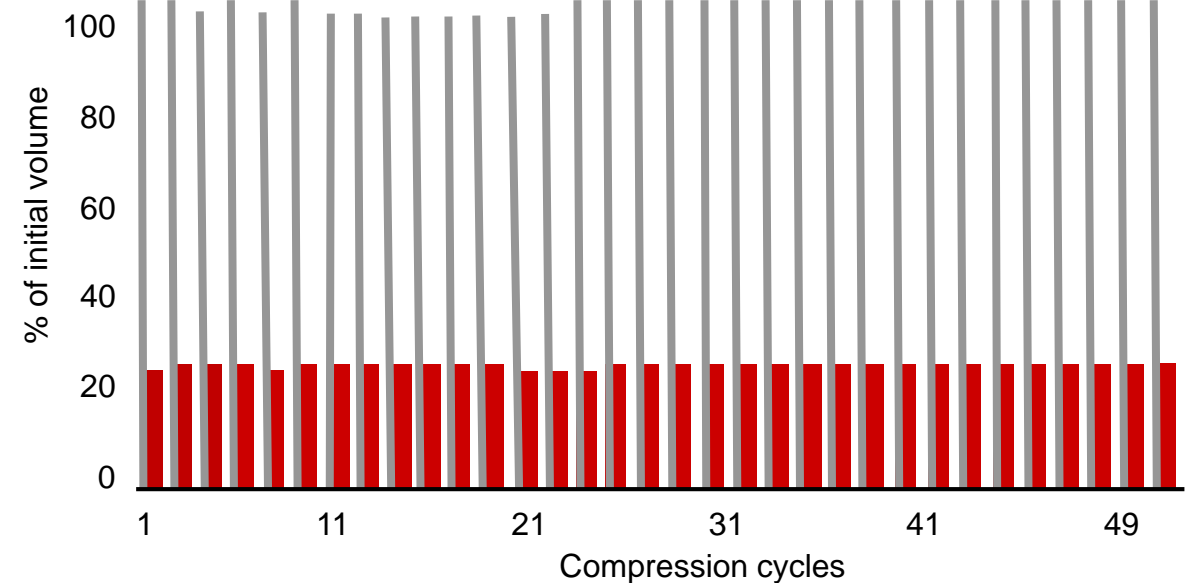

In vitro compression test shows volume maintenance upon repeated compression of the hydrated matrix¹. Gray bars represent the regained/ initial volume of the matrix; red bars show the compressed volume.

1. Olde Damink L, et al. 52nd Annual Conference of the German Society for Biomedical Engineering; Sep 26-28, 2018; Aachen.

Note: mucomaix® (Matricel GmbH) is distributed as creos mucogain since October 2018.



creos™ mucogain: soft-tissue regeneration

creos mucogain is suitable for **various soft-tissue indications**, such as soft-tissue volume augmentation and guided tissue regeneration procedures in recession defects for root coverage*.

In submerged healing indications, creos mucogain **substitutes the need for a second surgical site** to harvest soft tissue. This avoids the post-operative pain, bleeding, swelling and infection associated with grafting procedures¹⁻³, as well as limits on the quantity of harvestable soft tissue for grafting.

*See Instructions For Use for full prescribing information, contraindications, warnings and precautions.

1. Harris RJ et al. Int J Periodontics Restorative Dent. 2005 Oct;25(5):449-59.

2. Griffin TJ et al. J Periodontol. 2006 Dec;77(12):2070-9.

3. Aguirre-Zorzano LA et al. J Clin Exp Dent. 2017 Dec 1;9(12):e1439-e1445.

creos™ mucogain: no clinical complications in initial clinical use

1. Montero E, et al. 27th Annual Scientific Meeting of the European Association for Osseointegration; Oct 11-13, 2018; Vienna.
2. Wessing B, Vasilic N. Clin Oral Implants Res 2014;25(s10):342.
3. Wessing B, et al. German Clinical Trials Register DRKS00015213.
Note: mucomaix® (Matricel GmbH) is distributed as creos mucogain since October 2018.

Prospective case series

12 patients, immediate implant placement with simultaneous soft tissue augmentation and immediate provisionalization, 1-year follow-up¹

Study outcome

No significant adverse events related to creos mucogain were observed during the healing phase and up to the last visit.

Retrospective case series

7 patients, root coverage in 1 patient and biotype thickening in 6 patients (for immediate implant placement, before second-stage surgery, or during second-stage surgery); 4-week follow-up²

Study outcome

Uneventful healing reported for all cases.

Study expanded to retrospectively assess 45 patients treated with mucogain. Currently ongoing.³

References

- Aguirre-Zorzano LA, García-De La Fuente AM, Estefanía-Fresco R, et al. Complications of harvesting a connective tissue graft from the palate. A retrospective study and description of a new technique. J Clin Exp Dent 2017;9(12):e1439-e1445.
- Boekema B, Vlig M, Olde Damink L, et al. Effect of pore size and cross-linking of a novel collagen-elastin dermal substitute on wound healing. J Mater Sci Mater Med 2014;25(2):423-433.
- Griffin TJ, Cheung WS, Zavras AI, et al. Postoperative complications following gingival augmentation procedures. J Periodontol 2006;77(12):2070-2079.
- Harris RJ, Miller R, Miller LH, et al. Complications with surgical procedures utilizing connective tissue grafts: a follow-up of 500 consecutively treated cases. Int J Periodontics Restorative Dent 2005;25(5):449-459.
- Heschel I, et al. 2002. Method for producing porous structures. US patent 6,447,701 B1.
- Olde Damink L, Heschel I, Leemhuis H, et al. Soft tissue volume augmentation in the oral cavity with a collagen-based 3D matrix with orientated open pore structure. Paper presented at: 52nd Annual Conference of the German Society for Biomedical Engineering; Sep 26-28, 2018; Aachen.
- Montero E, Sanz-Martin I, Sanz-Sanchez I, et al. Volumetric changes in the buccal contour after immediate implant placement and provisional restoration together with a soft tissue substitute. A prospective case series. Abstract presented at: 27th Annual Scientific Meeting of the European Association for Osseointegration; Oct 11-13, 2018; Vienna.
- Wessing B, Vasilic N. Soft tissue augmentation with a new regenerative collagen 3-d matrix with oriented open pores as a potential alternative to autologous connective tissue grafts. Clin Oral Implants Res 2014;25(s10):342.
- Wessing B, et al. Soft tissue volume augmentation at single implant sites with a collagen based 3D matrix with oriented open pore structure: A retrospective analysis of 45 consecutive cases. German Clinical Trials Register DRKS00015213.

© Nobel Biocare Services AG, 2019. All rights reserved. Distributed by: Nobel Biocare. Legal Manufacturer: Matricel GmbH, Kaiserstrasse 100, 52134 Herzogenrath, Germany. Nobel Biocare, the Nobel Biocare logotype and all other trademarks are, if nothing else is stated or is evident from the context in a certain case, trademarks of Nobel Biocare. Please refer to nobelbiocare.com/trademarks for more information. Product images are not necessarily to scale. All product images are for illustration purposes only and may not be an exact representation of the product. Disclaimer: Some products may not be regulatory cleared/released for sale in all markets. Please contact the local Nobel Biocare sales office for current product assortment and availability. For prescription use only. Caution: Federal (United States) law restricts this device to sale by or on the order of a licensed clinician, medical professional or physician. See Instructions For Use for full prescribing information, including indications, contraindications, warnings and precautions.